

NHS Grampian Joint Wound Care Formulary

Product:

365 Film (Adults only, for paediatrics [see Tegaderm® Film monograph](#))

Product description:

365 Film from 365 Healthcare is a sterile, latex free, adherent transparent dressing. It is composed of an ultra-thin polyurethane film coat sandwiched between two protective liners. It is breathable with a high moisture vapour transfer rate. If applied correctly it provides a showerproof and bacterial barrier, and may be left in place for up to seven days.

Indications for use:

365 Film can be used on wounds where there is no exudate or light levels of exudate. It is suitable for cuts and abrasions; clean, closed, post-operative wounds; superficial pressure areas and leg ulcers; superficial burns; donor sites.

If signs of clinical infection are observed the dressing should be removed.

Contraindications:

Not indicated for patients with friable skin or paediatric patients.

Not indicated for use on full-thickness burns, deep cavity wounds, exuding or infected wounds.

Not indicated for use on patients with known sensitivity to the dressing or its components.

Precautions:

The bacterial barrier property of the dressing is compromised if the dressing is punctured, not adhered to the skin fully or exudate escapes from under the dressing.

Product type:

Film

Product function:

Primary dressing

Secondary dressing

Adhesive

Wound type:

Manages no exudate to low exuding wounds: epithelialisation, granulating, slough, necrosis.



Product	Manufacturer	Available sizes	Pack size
365 Film	365 Healthcare	10cm x 12cm	50
		10cm x 15cm	50
		15cm x 20cm	25