

NHS Grampian Joint Wound Care Formulary

Product:

Atrauman®

This is the first-line non-adherent dressing within the Grampian Wound Care Formulary.

Product description:

Atrauman® is knitted polyester dressing impregnated with triglycerides (neutral fats). Water-repellent polyester filaments and the smooth surface structure of the dressing allows drainage of exudate onto a secondary dressing and prevents adhesion to wound by preventing granulating tissue penetrating through the dressing. The secondary dressing can be a low adherent or foam dressing determined by wound exudate levels. The dressing can be left in place for up to seven days depending on the wound exudate levels

Indications for use:

Atrauman® is indicated for use as a primary wound contact dressing in a wide variety of wounds, including infected wounds.

Contraindications:

Nil of note

Precautions:

Requires a secondary dressing

Product type:

Non adherent

Product function:

Primary dressing

Wound type:

Not an exudate manager but can be used for low to highly exuding wounds: necrosis, slough, granulating, epithelialisation



Product	Manufacturer	Available sizes	Pack size
Atrauman® flat dressing	Hartmann	5cm x 5cm	10 community
		5cm x 5cm	50 hospital
		7.5cm x 10cm	10 community
		7.5cm x 10cm	50 hospital
		10cm x 20cm	30
		20cm x 30cm	10