

## NHS Grampian Joint Wound Care Formulary

**Product:**  
INADINE®

### Product description:

INADINE® is a knitted viscose primary dressing impregnated with povidine iodine ointment 10%. In the presence of wound exudate the povidine iodine, a potent anti-microbial agent is released, the dressing becomes inert when it loses colour and requires to be changed. Requires a secondary dressing such as a low adherent dressing.

Also see the statement on the use of [topical antimicrobial therapy](#).

### Indications for use:

INADINE® is indicated for the prophylaxis and treatment of infection in wounds such as minor burns and superficial skin-loss injuries.

### Contraindications:

Not indicated for use on patients with known iodine hypersensitivity.

Not indicated for use on patients with severe renal impairment, on the newborn child, in pregnancy or in lactating mothers, as there are possible effects of elevated serum iodine levels on the foetus or neonate.

### Precautions:

The dressing is unlikely to retain significant levels of anti-microbial activity if left in position for longer than two days.

No more than four dressings should be applied at one time.

Povidine iodine may be absorbed particularly if large wounds are treated.

INADINE dressing should be used with caution on patients with thyroid disease.

### Product type:

Antimicrobial Iodine

### Product function:

Primary dressing

Low adherent

Antimicrobial

### Wound type:

Not an exudate manager but can be used for low to highly exuding wounds: necrosis, slough, granulating, epithelialisation.



Product	Manufacturer	Available sizes	Pack size
INADINE®	Systagenix	5cm x 5cm	25
		9.5cm x 9.5cm	25